

# Applying Hanton Shoes is Easy!



**Step 1:** On a hard, clean surface trim & shape the foot as normal.

**Step 2:** Rasp the sides of the hoof where the tabs & blisters will go. Make sure the bottom & sides of the hoof are VERY CLEAN.



**Step 3:** Remember, cleanliness is the key to success! Always place the foot down on a clean surface such as a towel or mat. The cleaner the hoof surface, the better the glue will hold.

**Step 4:** Remove one layer of paper backing from the double sided tape and apply it to the front of the shoe (from the tabs forward) on the side that will be placed against the horse's foot.



**Step 5:** Shape the Hanton Horseshoe to fit the hoof (as with any standard shoe), making sure the shoe is flat to the bottom of the hoof.



**Step 6:** For extra glue adherence, lightly grind or sand the outside of the tabs.

**Step 7:** Remove the paper backing from the double sided tape.



**Step 9:** With the hoof on the ground, use a hammer to tap the special tabs into place until flush with the hoof wall.



**Step 11:** Stick the Hanton blisters onto the outside of the shoe around the tabs, aligning the shape of the blister as closely as possible with the shape of the tab. Be sure to press all edges of the Hanton blister firmly into place over the hoof & shoe.



**Step 8:** Place the shoe on the horse's hoof and set the hoof on the ground with the shoe in place.



**Step 10:** Remove the paper backing from the Hanton blisters to expose the adhesive.



**Step 12:** Attach the mixing tube to the glue dispenser, then dispense a small amount of glue through the mixing tube to ensure the glue is mixing at the correct ratio before filling the blister.

**Step 13:** Place the tip of the mixing tube in the hole in the Hanton blister and fill each blister with glue until entirely full. Wipe away any excess glue.



**Step 14:** For best results, wrap the entire hoof in stretch wrap & set the hoof down to cure. The horse should remain weight-bearing and non-moving until the glue has set. The foot may feel warm to the touch - this is normal.

**Step 15:** After several minutes and once the hoof is cool and the glue has set, remove the plastic wrapping.



**Step 16:** Peel away the perforated sides of the Hanton blister, leaving the main portion of the blister covering the tab & glue. A rasp may be used to remove any small amounts of excess glue around the blisters.

**In a matter of minutes, this horse is done & ready to go!**

