



## How to Size

Soft-Ride Comfort Boots are available in 22 different sizes, and our Gel Orthotics come in 22 different sizes and six different densities, so that makes getting the correct size important. Our sizing video and horse boot sizing chart will help your horse get the proper fit.

Correctly measuring the hoof will ensure a comfortable fit for your horse and help prevent premature wear of the boot. Watch the video and carefully follow the steps below to ensure accurate measurements.

### Important Sizing Notes

We recommend these guidelines when measuring your horse and using Soft-Ride boots:

1. Measure your horse's hooves shortly after they have been trimmed. This will help ensure that the boot is not too big.
2. It's important to keep your horse's hooves trimmed. This will help ensure continued proper fitment and help prevent damage to the boot.
3. Measure each hoof to check for discrepancies in hoof size. You may find it necessary to order a different size boot for each foot.
4. When you receive your Soft-Ride boots, remove the Gel Orthotic and hold it up to the bottom of the hoof. The Gel Orthotic should overlap the hoof by up to an eighth inch on all sides to allow for some growth before the hoof is trimmed again. If the foot overlaps the Gel Orthotic in length or width, then the boot is too small.


## Sizing Instructions

We recommend these guidelines when measuring your horse and using Soft-Ride boots:




### STEP 1

Place the hoof squarely on a sturdy piece of paper or cardboard (such as a manila folder).

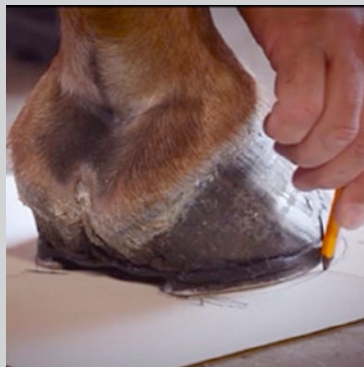

### STEP 2

It's important to keep your horse's hooves trimmed. This will help ensure continued proper fitment and help prevent damage to the boot.

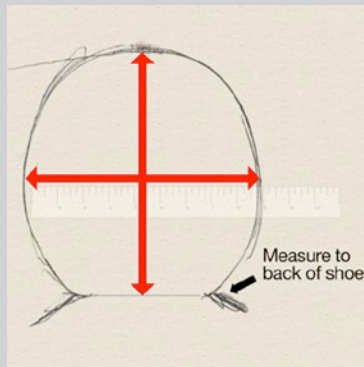

### STEP 3

Measure each hoof to check for discrepancies in hoof size. You may find it necessary to order a different size boot for each foot.



### STEP 4

Hold the gel orthotic up to the bottom of the hoof. The gel should overlap the hoof by up to an eighth of inch on either side. This allows for some growth before the hoof is trimmed again. If the foot overlaps the Gel Orthotic in length or width, then the boot is too small.



# Sizing Chart

Soft-Ride has added long, narrow, and long-narrow orthotics in select sizes. When reordering, specify the orthotic size. Additionally, specialty pairs of orthotics can be purchased in these new sizes.

| Size           | Width          | Length         |
|----------------|----------------|----------------|
| 2              | 6 5/8" / 168mm | 6 1/2" / 164mm |
| 2 Narrow*      | 5 3/4" / 146mm | 6 3/8" / 162mm |
| 3 Long Narrow* | 5 3/4" / 146mm | 6 3/4" / 171mm |
| 3              | 6 3/8" / 162mm | 6" / 152mm     |
| 3 Narrow*      | 5 1/4" / 133mm | 6" / 152mm     |
| 4              | 5 3/4" / 146mm | 5" / 127mm     |
| 4 Narrow       | 5" / 127mm     | 5 1/4" / 133mm |
| 5 Long         | 5 3/4" / 146mm | 5 1/2" / 140mm |
| 5 Long Narrow* | 4 3/4" / 121mm | 5 3/4" / 146mm |
| 5              | 5 3/4" / 146mm | 5 1/4" / 133mm |
| 5 Narrow       | 4 3/4" / 121mm | 5 1/4" / 133mm |

| Size           | Width          | Length         |
|----------------|----------------|----------------|
| 6 Long         | 5 3/8" / 136mm | 5 1/4" / 134mm |
| 6 Long Narrow  | 4 3/4" / 121mm | 5 1/4" / 133mm |
| 6              | 5" / 127mm     | 4 1/2" / 114mm |
| 6 Narrow       | 4 1/4" / 108mm | 4 3/4" / 121mm |
| 7 Long         | 4 7/8" / 124mm | 5" / 127mm     |
| 7 Long Narrow* | 4 3/8" / 110mm | 5" / 127mm     |
| 7              | 4 3/8" / 110mm | 4" / 102mm     |
| 7 Narrow       | 4" / 102mm     | 4" / 102mm     |
| 8              | 3 3/4" / 96mm  | 3 3/4" / 96mm  |
| 9              | 3 1/4" / 83mm  | 3 1/4" / 83mm  |
| 10             | 2 3/4" / 70mm  | 2 1/2" / 64mm  |

Approximately boot orthotic size comparisons with Standard US "Keg" shoes

| Orthotic Size     | #2 | #3LN | #3 | #5L | #5LN | #6L | #6LN | #7L | #7L | #7  |
|-------------------|----|------|----|-----|------|-----|------|-----|-----|-----|
| US Keg Horse Size | 5  | 4    | 3  | 2   | 1.5  | 1   | .05  | 0   | 00  | 000 |

\*Soft-Ride has added long and long narrow sized orthotics. Narrow orthotics accommodate ill shaped hoofs. When reordering, specify the orthotic size.

For a narrower hoof needing an orthotic with less width; narrow size orthotics are available as standard in size #2 Narrow, #3 Narrow, #5 Narrow, #5 Long Narrow, #6 Long Narrow, #7 Long Narrow, and #7 Narrow. Specialty pairs of orthotics can be purchased additionally in these narrow sizes.

## Contact

Soft-Ride, Inc.  
626 Grand Ave.  
Bacliff, TX 77518

US: 855.763.8743  
Int: 281.334.2576  
[Sales@softrideboots.com](mailto:Sales@softrideboots.com)

